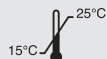
septodont	SEPTODONT ARTWORK CODE: M05 120 144 00 00	COLOR(S)	3	NOT PRINTABLE - AREA FOR VARIABLE DATA STICKER  The following information will be printed on line during manufacturing process : [LOT] + variable data text  + variable data text UDI QR code + variable data text
	CREATION DATE: 30/01/25	  		
	MODIFICATION DATE: 5/02/25			
	DIMENSIONS : 156 mm x 125 mm			
	GRAPHIC DESIGNER: Jean			

REF 248866

Largal L

EDTA 17%

For the instrumentation of Root
Canals and Smear Layer Removal

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en - Description: EDTA solution 17% (ethylenediaminetetraacetic acid) is an amino acid compound, a powerful chelating agent to remove inorganic debris (smear layer) and lubricate the canals in endodontics. EDTA is used in 17% as neutralized solution to loosen up calcifications, remove smear layer, allow instrumentation (canals shaping) and debidement in a tight/calcified root canal towards the apex. For professional use only. **Instructions for use:** Irrigate with EDTA solution as needed for debidement during instrumentation. 1. Fill syringe with EDTA solution (5-10 ml recommended). 2. Irrigate as desired. 3. Remove and dispose of syringe and irrigating tip. **Precautions:** 1. Read MSDS for material handling and safety information. 2. Irrigation should not be irritant for the apical area. **Warnings:** 1. Always replace cap after use. 2. Keep out of reach of children. 3. Avoid contact with skin, eyes and inhalation. **Storage:** Store in a room temperature, dry area, away from heat and direct sunlight.

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