
septodont	SEPTODONT ARTWORK CODE: M05 132 143 00 00	COLOR(S)	3	NOT PRINTABLE - AREA FOR VARIABLE DATA STICKER
	CREATION DATE: 31/01/25	Pantone	Pantone	The following information will be printed on line during manufacturing process : [LOT] + variable data text [] + variable data text [] UDI QR code + variable data text
	MODIFICATION DATE: 3/02/25	185	427	
	DIMENSIONS : 156 mm x 125 mm	Black		
	GRAPHIC DESIGNER: Jean			

REF 248865

R4 Ultra

2% Chlorhexidine Gluconate Solution

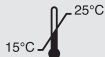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en - Description: R4 Ultra is a chlorhexidine digluconate solution. It is intended for the irrigation and debridement of root canals during and after instrumentation in endodontic treatment. Remove smear layer with appropriate irrigant such as EDTA solution or Citric Acid 12.5%. For professional use only. **Instructions for use:** 1. Use 17% EDTA as a pre-rinse in the canal to remove smear layer. 2. Rinse with Sodium Hypochlorite solution (NaOCl). 3. Rinse out the NaOCl with sterile water or saline solution. 4. Fill syringe with R4 Ultra (10 ml recommended). 5. Continuously irrigate for a minimum of 2 minutes (10-20 ml per canal recommended). 6. Remove and dispose of syringe and irrigating tip. **Precautions:** 1. Read MSDS for material handling and safety information. 2. Irrigation should not be irritant for the apical area. 3. Not to be used as an oral rinse. **Warnings:** 1. Products containing chlorhexidine must not be mixed with products containing sodium hypochlorite to prevent formation of a reddish-brown low solubility precipitate. 2. Keep out of the reach of children. 3. Avoid contact with skin and eyes. **Storage:** Store at room temperature, dry area, away from heat and direct sunlight.

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